



Klonidin	Ibuprofen	Oxymetazolin	Trazodon
Kodein	Imipramin	Oxymorfon	Trimethobenzamid
Kortizon	Iproniazid	Papaverin	Triamteren
(-)-Kofein	(-)-isoproterenol	Pemoline	Trifluoperazin
Kreatinin	Isoxsuprin	Penicilin-G	Trimethoprim
Cyklobarbitál	Kanamycin	Pentazocin	Trimipramin
Cyklobenzaprin	Ketamin	pentobarbital	Tryptamin
Deoxykortikosteron	Ketoprofen	Perfenazín	d,l-Tryptofan
R(-)Deprenyl	Labetalol	Fencyklidin	Tyramin
Dextromethorfan	Levorphanol	Fenelzín	d,l-tyrosin
Diazepam	Lidokain	feniramin	Kyselina močová
Diclofenac	Lindane	Fenobarbital	Verapamil
Dicyklomin	(Hexachlorcyklohexan)	Fenothiazin	Digoxin
Difunisal	Loperamid	Phentermine	Uhlíčan lithný
I-Fenylefrin			

#### 【BIBLIOGRAFIE】

1. International Narcotics Control Board.Report of the International Narcotics Control Board for 2009[R].New York: UN. 2010
2. Lane JC, Tennison MB, Lawless ST, et al.Movement disorder after withdrawal of fentanyl infusion.J Pediatr. 1991, 119 (4): 649-651
3. Dominguez KD, Lomako DM, Katz RW. et al. Opioid withdraw in critically ill neonates.Ann Pharmacotherm, 2003, 37 (4): 473-477
4. European Monitoring Centre for Drugs and Drug Addiction.Annual Report 2009[R].Lisbon: EMCDDA, 2010

#### Index of Symbols

	Pozor		Testy na sadu		Oprávněný zástupce
	Pouze pro použití v diagnostice „in vitro“		Použitelné do		Nepoužívejte opakovaně
	Skladujte při teplotě 2-30 °C		Číslo šarže		Katalogové číslo
	Nepoužívejte, pokud je obal poškozený		Výrobce		Přečtěte si návod k použití



**Hangzhou AllTest Biotech Co.,Ltd.**  
 #550,Yinhai Street  
 Hangzhou Economic & Technological Development Area  
 Hangzhou, 310018 P.R. China  
 Web: www.alltests.com.cn Email: info@alltests.com.cn



**MedNet EC-REP GmbH**  
 Borkstrasse 10  
 48163 Muenster  
 Germany

Číslo:  
 Datum účinnosti: 2023-10-16