Languages:

ENG

CZ

SK

# USER MANUAL

**BREATHALYZER**  
*electrochemical*

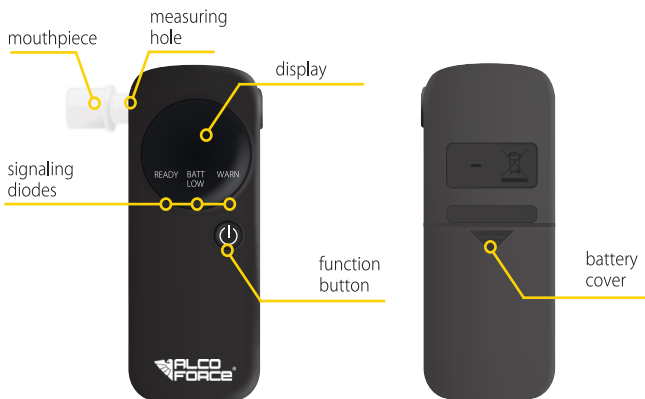
**PROFESSIONAL**  
ver 2.2

Dear Customer, congratulations on your choice! We invite you to read the user's manual, so that the purchased device serves you in best possible way.

## SCHEME & SPECS

ENG

testing range	0,00 ~ 4,00 ‰
precision	+/- 0,07‰ at 0,5 ‰ in 25°C
sensor	platinum electrochemical
power supply	2 x AAA 1,5V alkaline
weight	47 x 114 x 24 mm
dimensions	84g ( with batteries)
re-calibration	every 12 months or 600 tests
mouthpiece	required - universal AlcoForce
other	sensor control, battery level control, airflow control, temperature protection

## PREPARING THE DEVICE FOR WORK

Insert the batteries into the device, observing correct polarity [ATTENTION - only alkaline batteries should be used with the device!!!].

### ATTENTION! BEFORE MEASUREMENT!

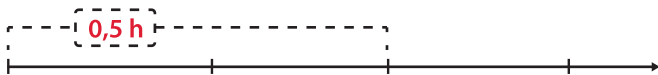
ENG

Rinse your mouth after drinking the alcohol and wait 30 minutes. This amount of time is crucial for the alcohol to absorb into your bloodstream and to reduce the alcohol vapours in your mouth and saliva. For the correct results, do not eat, drink or smoke cigarettes for min. 30 minutes before taking the measurement. Do not take measurements in strong winds, in air-conditioned rooms, or in polluted air. Do not use immediately after hand disinfection or in the rooms with present disinfectants.

Many factors may interfere with the measurement results (heavy environmental pollution, smoking, various gas vapors, wind, air conditioning, alcohol vapors coming from disinfectants), hence the results presented by the breathalyzer are only estimates. The manufacturer, importer and seller do not bear any responsibility for the correctness of the results and potential damage caused by improper use of the breathalyzer. In line with the Polish law, the device cannot be used as an evidence breathalyzer.

## PERFORMING THE MEASUREMENT

End smoking, eating and drinking

Rinse your mouth

Switch on the device

Take 3 measurements at 1 minute intervals. Choose the highest score.

During the measurement, follow the information on the breathalyzer display and the instructions.

Insert the mouthpiece into the measuring hole (\* 1). Turn on the device by pressing the function button on the main panel (\* 2).

After starting the device, you will hear a single beep. The display will show three eight notes and all the indicator lights will turn on. Then the message **On** will appear. The breathalyzer will display the number of measurements already performed and start preparing for the test, counting from 150 to 0.

The device is ready to take an air sample. Take a deep breath and blow into the mouthpiece for approx. 6 sec. The device will emit an intermittent beep while the sample is being taken.

End of the sound signal. The breathalyzer analyzes the collected sample.

After analyzing the sample, the breathalyzer will display the result. The measurement result from 0.00 to 0.50 ‰ is also signaled by the green diode. If the result exceeds 0.50 ‰ the red "WARN" diode will light up.

After about 20 seconds the result will disappear and the breathalyzer will display the message **OFF** and shut off automatically.


## HOURLY ALCOHOL ELIMINATION

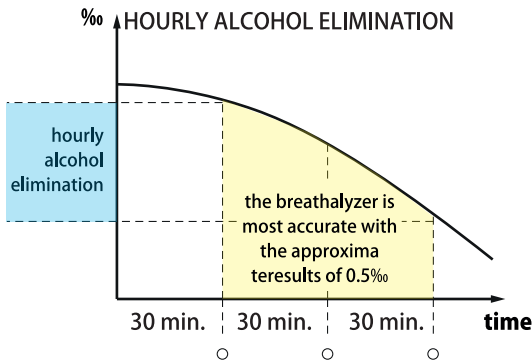The alcohol level in blood and exhaled air changes over time. At first, it increases during the absorption of alcohol into the blood, then decreases during its elimination (burning) in the body.

A breathalyzer can help you estimate which phase you are in: alcohol concentration increase or elimination. To obtain a reliable result, take measurements when you no longer intend to consume alcohol.

We suggest to make min. 3 measurements at equal 30 minute intervals and to record the individual results. Comparison of these results will give an answer to the question in which phase you are - using 3 decreasing results obtained in equal intervals it is possible to estimate when the body will get rid of alcohol.

3 decreasing measurements taken at intervals of 30 minutes.

Provided air sample is too small. Perform the measurement again providing a strong and steady exhalation.

Low voltage. The device turns off. Replace the batteries.

The sensor calibration is required. Contact the service center. More information available from the page 8

Measurement result above 4.00 %. The measurement was probably taken immediately after drinking alcohol. Wait 30 minutes before the next test. If the message persists, please contact the service center.

The ambient temperature outside the device operating temperature range (5° -40°C). The breathalyzer switches off.

If you have the impression that your breathalyzer is not working properly, we recommend to check the following:

1. The use of old or improper batteries may affect the accuracy of the results. Use only high-quality alkaline batteries. Do not use rechargeable batteries.
2. Decalibration of the breathalyzer. Each breathalyzer should be calibrated in accordance with its technical specification. For 12 months from the purchase of the breathalyzer, we will perform the calibration service for FREE!!! For more information, please visit [alcoforce.com](http://alcoforce.com).
3. Everyone has a different ability to metabolize alcohol, and this ability may change from time to time depending on various factors. If your Professional breathalyzer is regularly calibrated according to the service interval and you use it according to the instructions and the following three results are similar, it is most likely working properly. To read more about the factors affecting the results of the breathalyzer measurements please visit: [alcoforce.com](http://alcoforce.com).

**THE SELLER DOES NOT ACCEPT SERVICE REQUESTS**  
**service requests are considered by the central service**  
**AlcoForce / Work and Road Safety Corporation**

## STORAGE AND GENERAL REMARKS

Do not eat, drink or smoke for 30 minutes before taking the measurement.

Store in a dry place, avoid direct exposure to sunlight and temperatures outside the range of 0 to 40 °C. Do not keep in the car!

Do not take the measurement immediately after drinking alcohol. Blowing into a breathalyzer immediately after drinking alcohol may cause damage to the sensor and result in improper measurements.

ENG

Do not disinfect or clean the device with alcohol and other disinfectants. Do not touch the breathalyzer for at least 30 minutes after hand disinfection.

Do not store the breathalyzer close to volatile substances, including alcohol. Do not blow fluids and fumes into the sensor and do not immerse the device in fluids. Do not clean the device with detergents.

Service the device regularly or as soon as it displays a service message. Within 12 months from the date of purchase, the service is completely free of charge\* - use it. Avoiding servicing will negatively affect the lifetime and precision of the breathalyzer!

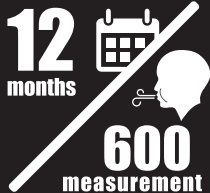
\*CAUTION. Applies to first user purchase.

**UWAGA**

**Calibration is a term inextricably linked with EVERY BREATHALYZER.** This means that every 12 months, the breathalyzer must be reconfigured so that it correctly interprets the readings of the wearing-out sensor. The appropriate messages displayed on the device indicate the need for calibration (page 5) as well as the results that differ significantly from reality, e.g. 0.5 ‰ despite the lack of alcohol consumption.


**The necessity of calibration is not a disadvantage of the breathalyzer and is not subject to complaint.** Please do not expect that a breathalyzer that is not calibrated regularly will show the correct results. Calibrations after the free service period are chargeable and can be made at the premises of ALCOFORCE / WORK AND ROAD SAFETY CORPORATION (also by mail order) or at another authorized breathalyzer service point. **Additional information can be found in the Warranty Card on page 9 .**

**12**  
months

**600**  
measurement

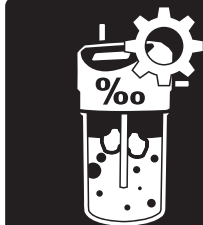Calibration in this model should be performed every **600 measurement** or **12 months** (regardless of the number of measurements performed). Regular calibration is necessary to maintain proper operation of the device and extends the life of the sensor.

**Note** - calibration of this breathalyzer within 12 months from the date of purchase is **FREE OF CHARGE** at the premises of AlcoForce and Work and Road Safety Corporation of other authorized service points.  
**More information can be found in the Warranty Card.**

- ENG**
1. Work and Road Safety Corporation PLC guarantees the correct operation of the breathalyzer for 12 months from the date of sale, provided that it is used in accordance with its intended use and in accordance with the detailed rules set out in the manual.
  2. The warranty liability includes free repair during the warranty period, except for the situations specified in the user manual below.
  3. If the breathalyzer is out of order or requires repair, contact an authorized service center (page 6)
  4. Detailed instructions on the complaint procedure can be found on page 10.
  5. The warranty does not cover components subject to wear and the device decalibration (see the manual).
  6. The warranty does not cover the following faults:
    - a) caused by improper operation or use of the breathalyzer inconsistently with intended use,
    - b) caused by unauthorized modification of the product, or resulting from repairs and service activities carried out by unauthorized persons,
    - c) mechanical damage
    - d) resulting from improper storage, transport or use in conditions that prevent proper operation
  7. The warranty will not be valid if the proof of purchase is not provided.
  8. Returning a device that does not meet the warranty conditions may require the Customer to cover the costs of transport.
  9. An unfilled Warranty Card will be filled in by a service technician during the first service of the device.
  10. The warranty rights are exercised in Czech Republic in the facilities of Work and Road Safety Corporation s.r.o. and cover the repair of the device or its replacement with the new one.
  11. The warranty does not exclude, limit or suspend any rights of the Purchaser stemming from the regulations regarding a warranty for defects in sold goods.

***Before contacting the service, please check if the solution to your problem is not already provided in the user manual.***

**Serial No.:**

**Place and date of sale:**

**Seller Stamp:**

Do you think your breathalyzer needs maintenance? Register online and you will be able to send the breathalyzer to us completely free of charge!

**The online ticket system is nothing but benefits!**

**CAUTION - CALIBRATION OF THIS BREATHALYZER IN THE PERIOD OF 12 MONTHS FROM THE DATE OF PURCHASE IS FREE OF CHARGE (ONLY AT THE AUTHORIZED SERVICE).**

ENG

Free shipping of the breathalyzer to the service!

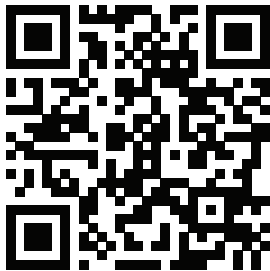
Saving your time.  
You will get everything done in a couple of minutes.

Faster service delivery - we treat online applications as a priority.

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