Languages:

ENG

CZ

SK

# USER MANUAL

# RAPTOR AT 7000

ver 2.0

**BREATHALYZER**  
*electrochemical*

SCHEME & SPECS	1
PREPARING THE DEVICE FOR WORK	2
POWER SUPPLY	3
BUTTON PANEL	3
PERFORMING MEASUREMENT - AUTOMATIC MODE	4
PERFORMING MEASUREMENT - PASSIVE MODE - MANUAL	5
USER MENU	6
LED INDICATOR	8
TROUBLESHOOTING	9
STORAGE AND GENERAL REMARKS	9
CALIBRATION	10
WARRANTY CARD	10
SERVICE, CALIBRATIONS, COMPLAINTS	11

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Dear Customer, congratulations on your choice! We invite you to read the user's manual, so that the purchased device serves you in best possible way.

Congratulations on choosing the **ALCOFORCE RAPTOR** breathalyzer.

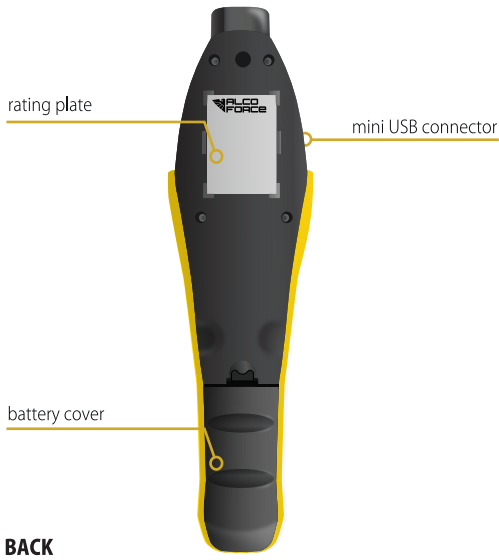
ALCOFORCE RAPTOR is the latest device for rapid mass screening for the presence of alcohol in exhaled air. The device utilizes the patented Fast Scan technology, based on a platinum electrochemical sensor. Please read the following instructions carefully and adhere to the recommendations to ensure prolonged, trouble-free, and efficient usage.

**CAUTION:** The ALCOFORCE RAPTOR breathalyzer is intended for preliminary verification of alcohol presence in the breath of the tested person. The results are approximate and may differ from the results obtained from a mouthpiece breathalyzer.

## SCHEME & SPECS

testing range	0,0 ~ 2,0 mg/l
precision	+/- 0,05‰ at 0,5 ‰ in 25°C
sensor	platinum electrochemical
power supply	4 x AA 1,5V alkaline
weight	68,5 x 284 x 74 mm
dimensions	399 g ( with batteries)
re-calibration	every 12 months or 4000 tests
mouthpiece	no
other	measurement memory, light signaling, advanced self-checking, short measurement time

ENG


**PREPARING THE DEVICE FOR WORK**

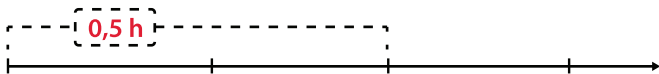Insert the batteries into the device according to the polarity [CAUTION - use only alkaline batteries!!!]

**ATTENTION! BEFORE MEASUREMENT!**

After consuming alcohol, rinse your mouth and wait for 30 minutes. This is the necessary time for alcohol to be absorbed into the bloodstream and reduce its vapors in the mouth and saliva. For accurate measurement, do not eat, drink, or smoke cigarettes for at least 30 minutes before the test. Do not perform measurements in strong winds, air-conditioned rooms, or polluted air. Do not use immediately after hand disinfection or in rooms with disinfectant fluids.

Several factors can interfere with measurement results (strong environmental pollution, cigarette smoke, various gas vapors, wind, air conditioning, alcohol vapors from disinfectant fluids), which is why the results presented by the breathalyzer are only approximate. The manufacturer, importer, and seller are not liable for the accuracy of the results or any damages caused by improper use of the breathalyzer. According to Polish law, the device cannot be used as legal evidence in alcohol testing.

End smoking, eating and drinking

Rinse your mouth

Switch on the device


Take 3 measurements at 1 minute intervals. Choose the highest score.

During the measurement, follow the information on the breathalyzer display and the instructions.


## POWER SUPPLY

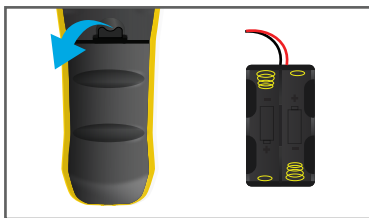Raptor is powered by 4 alkaline AA batteries with a voltage of 1.5V. To install the batteries, remove the rear battery cover and take out the battery tray located inside. Insert the batteries into the tray according to the marked polarity [**CAUTION - use only alkaline batteries!**].

Battery charge indicator:

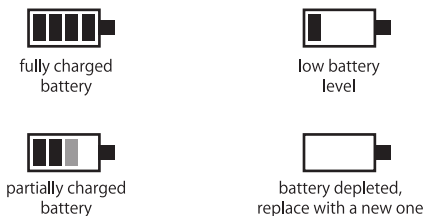fully charged  
battery

low battery  
level

partially charged  
battery


battery depleted,  
replace with a new one

ENG

## BUTTON PANEL

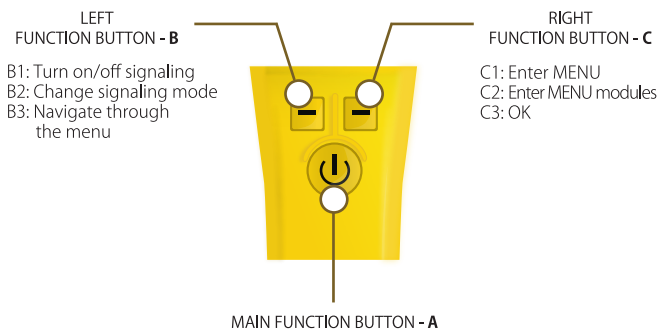Operation of the Raptor breathalyzer is carried out using 3 buttons. Each button is multifunctional - for convenience, the buttons will be labeled with the letters A, B, and C, while the individual functions of each button will be described with abbreviations: (A1), (A2), (A3), etc.

B1: Turn on/off signaling  
B2: Change signaling mode  
B3: Navigate through  
the menu

C1: Enter MENU  
C2: Enter MENU modules  
C3: OK

A1: ON/OFF  
A2: Passive measurement  
A3: Return from MENU to the main screen

Description of individual menu functions on page 6

Turn on the device by holding the ON/OFF function button (\*1). After about 2 seconds, the breathalyzer will prepare for measurement. The message **"BLOW"** indicates that you can start the measurement.

Take a deep breath and blow into the measurement opening for about 2 seconds. During the sample collection, the device will emit a continuous sound signal.

**2** End of the sound signal. The breathalyzer will analyze the collected sample.

**3** After the analysis, the breathalyzer will display the result:

**0,00 - 0,19 ‰**  
The display will illuminate green. The indicator light will also be green. No alcohol detected or the level is below 0,2‰.

**0,20 - 0,49 ‰**  
The display will illuminate yellow. The indicator light will be red. Alcohol detected.

**0,50 - 4,00 ‰**  
The display will illuminate red. The indicator light will also be red. High alcohol concentration detected.

The result will be displayed for about 6 seconds. After this time, the breathalyzer will prepare for the next measurement.

Turn on the device by holding the ON/OFF function button (\*1). After about 2 seconds, the breathalyzer will prepare for measurement. The message "BLOW" indicates that you can start the measurement.

Bring the breathalyzer close to the mouth of the person being tested / the liquid you want to test and press the ON/OFF button (\*2) to collect the sample.

2 End of the sound signal. The breathalyzer will analyze the collected sample.

ENG

3 After the analysis, the breathalyzer will display the result:

**0,00 - 0,19 ‰**

The display will illuminate green. The indicator light will also be green. No alcohol detected or the level is below 0,2‰.

**0,20 - 0,49 ‰**

The display will illuminate yellow. The indicator light will be red. Alcohol detected.

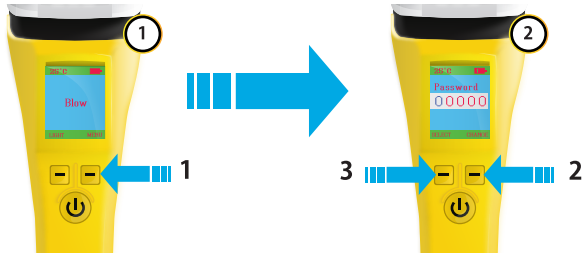
**0,50 - 4,00 ‰**

The display will illuminate red. The indicator light will also be red. High alcohol concentration detected.

The result will be displayed for about 6 seconds. After this time, the breathalyzer will prepare for the next measurement.

To access the user menu, press the right function button (\*1) when the breathalyzer is in standby mode (the screen displays "BLOW").

The menu may be protected by a 3 or 5-digit PIN password. The password can be set by the user or the service.


Enter the password digit by digit from 0 to 9 using the right function button (\*2) **CHANGE**. Confirm each entered digit with the left function button (\*3) **SELECT**. When all correct password digits are entered, the message **SELECT** will change to **OK**. At this point, confirm with the left function button

Navigation within the menu is done using the left function button with the message **SELECT**. To activate a selected module from the menu and confirm any changes made to the settings, press the right function button with the message **OK**. To return to the menu or measurement screen, press the ON/OFF button.

ENG

**Results**

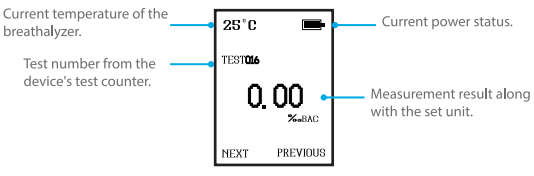
Review of data stored in the device's cache memory.

To view the results, press the right or left function button. Return to MENU using the ON/OFF button.

**Calibration**

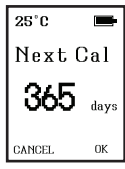Days counter for calibration and breathalyzer lock.


To return to MENU, press one of the three function buttons.

## Unit

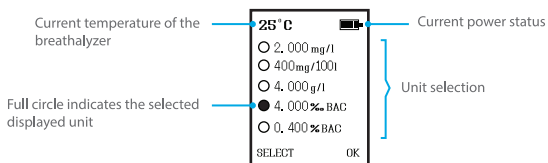
Change of displayed unit.

You can select units using the left function button. Confirm your selection with the right function button and the ON/OFF button. If you don't want to change the settings, choose the same setting and press the main ON/OFF button or the right function button. The breathalyzer will return to the MENU.

## Exhalation time

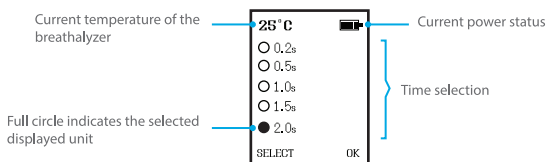
Changing the measurement time.

You can select the time using the left function button. Confirm your selection with the right function button and the ON/OFF button. If you don't want to change the settings, choose the same setting and press the main function button ON/OFF or the right function button. The breathalyzer will return to the MENU.

## Result display time

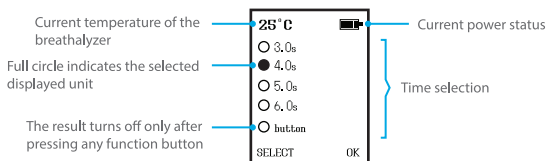
Changing the measurement display time.

You can select the time using the left function button. Confirm your selection with the right function button and the ON/OFF button. If you don't want to change the settings, choose the same setting and press the main function button ON/OFF or the right function button. The breathalyzer will return to the MENU.

## Language

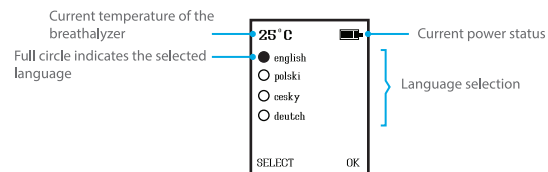
Change language in the menu.

You change the language with the left function button. Use the right function button and the ON/OFF button to confirm your selection. If you don't want to change the settings, choose the same option and press the main function button ON/OFF or the right button. The breathalyzer will return to the MENU.

## Password change

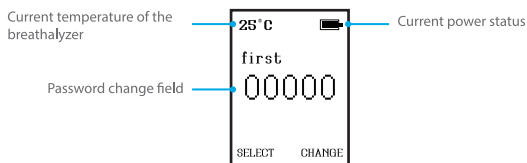Change the password for the user menu.

You can select individual digits with the right function button **CHANGE**. To change the password field, press the left function button **SELECT**. After entering the new PIN, accept it with the left function button **OK**. You must repeat this action to save the password. Return to MENU using the ON/OFF button.


**! THE DEVICE WILL AUTOMATICALLY TURN OFF AFTER 3 MINUTES OF INACTIVITY.**

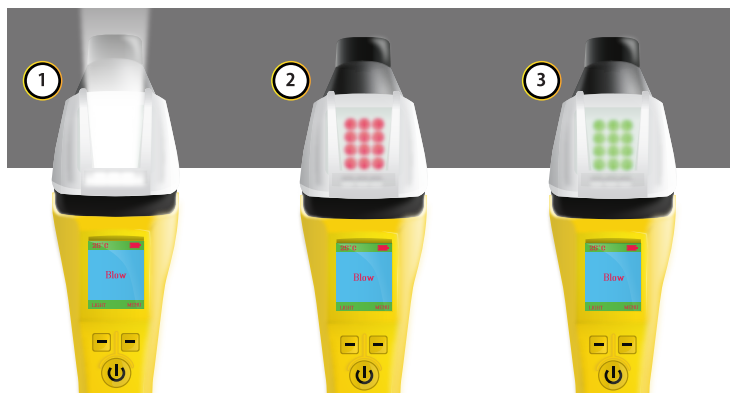ENG

**! PREVIOUS PRODUCTION UNITS OF THE DEVICE MAY HAVE A DIFFERENT SOFTWARE VERSION.**

## LED INDICATOR

Turn on the breathalyzer by holding the ON/OFF function button. To activate the LED indicator, press the left function button **LIGHT** when the message **BLOW** is displayed on the screen. Each press of the left function button changes the displayed light stage:

**White point light:** allows illuminating the face of the tested person in low ambient light or illuminating specific points chosen by the user.

**Pulsating red light:** warning light used, for example, by law enforcement agencies to control traffic.

**Pulsating green light:** warning light used, for example, by law enforcement agencies to control traffic.

The LED indicator will turn off after cycling through all light stages. Measurements can be taken while using the LED indicator.

If you have the impression that your breathalyzer is not working properly, we recommend to check the following:

1. Using incorrect or depleted batteries. Even freshly purchased batteries may turn out to be old and have reduced voltage. Remember to use only high-quality **alkaline batteries**.
2. Decalibration of the breathalyzer. Each breathalyzer should be calibrated in accordance with its technical specification. For 12 months from the purchase of the breathalyzer, we will perform the calibration service for **FREE!!!** For more information, please visit **alcoforce.com/cz**.
3. Everyone has a different ability to metabolize alcohol, and this ability may change from time to time depending on various factors. If your RAPTOR AT 7000 breathalyzer is regularly calibrated according to the service interval and you use it according to the instructions and the following three results are similar, it is most likely working properly. To read more about the factors affecting the results of the breathalyzer measurements please visit: **alcoforce.com/cz**.

## THE SELLER DOES NOT ACCEPT SERVICE REQUESTS

**service requests are considered by the central service AlcoForce / Work and Road Safety Corporation s.r.o.**

## STORAGE AND GENERAL REMARKS

Do not eat, drink or smoke for 30 minutes before taking the measurement.

Store in a dry place, avoid direct exposure to sunlight and temperatures outside the range of 0 to 40 °C. Do not keep in the car!

Do not take the measurement immediately after drinking alcohol. Blowing into a breathalyzer immediately after drinking alcohol may cause damage to the sensor and result in improper measurements.

Do not store the breathalyzer close to volatile substances, including alcohol. Do not blow fluids and fumes into the sensor and do not immerse the device in fluids. Do not clean the device with detergents.

Service the device regularly or as soon as it displays a service message. Within 12 months from the date of purchase, the service is completely free of charge\* - use it. Avoiding servicing will negatively affect the lifetime and precision of the breathalyzer!

\*CAUTION. Applies to first user purchase.

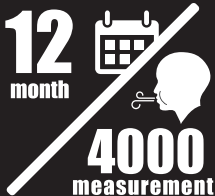
Do not disinfect or clean the device with alcohol and other disinfectants. Do not touch the breathalyzer for at least 30 minutes after hand disinfection.


**ATTENTION**

**Calibration is a term inextricably linked with EVERY BREATHALYZER.** This means that every 12 months, the breathalyzer must be reconfigured so that it correctly interprets the readings of the wearing-out sensor. The appropriate messages displayed on the device indicate the need for calibration (page 6) as well as the results that differ significantly from reality, e.g. 0.5 ‰ despite the lack of alcohol consumption.

**The necessity of calibration is not a disadvantage of the breathalyzer and is not subject to complaint.** Please do not expect that a breathalyzer that is not calibrated regularly will show the correct results. Calibrations after the free service period are chargeable and can be made at the premises of ALCOFORCE / WORK AND ROAD SAFETY CORPORATION (also by mail order) or at another authorized breathalyzer service point. **Additional information can be found in the Warranty Card.**

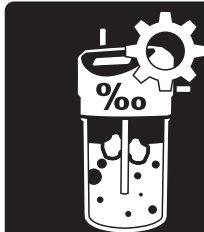Calibration in this model should be performed every **4000 measurement** or **12 months** (regardless of the number of measurements performed). Regular calibration is necessary to maintain proper operation of the device and extends the life of the sensor.

**Note** - calibration of this breathalyzer within 12 months from the date of purchase is **FREE OF CHARGE** at the premises of AlcoForce and Work and Road Safety Corporation of other authorized service points. **More information can be found in the Warranty Card.**

ENG

**WARRANTY CARD**

1. Work and Road Safety Corporation PLC guarantees the correct operation of the breathalyzer for 12 months from the date of sale, provided that it is used in accordance with its intended use and in accordance with the detailed rules set out in the manual.
2. The warranty liability includes free repair during the warranty period, except for the situations specified in the user manual below.
3. If the breathalyzer is out of order or requires repair, contact an authorized service center (page 11)
4. Detailed instructions on the complaint procedure can be found on page 11.
5. The warranty does not cover components subject to wear and the device decalibration (see the manual).
6. The warranty does not cover the following faults:
  - a) caused by improper operation or use of the breathalyzer inconsistently with intended use,
  - b) caused by unauthorized modification of the product, or resulting from repairs and service activities carried out by unauthorized persons,
  - c) mechanical damage
  - d) resulting from improper storage, transport or use in conditions that prevent proper operation
7. The warranty will not be valid if the proof of purchase is not provided.
8. Returning a device that does not meet the warranty conditions may require the Customer to cover the costs of transport.
9. An unfilled Warranty Card will be filled in by a service technician during the first service of the device.
10. The warranty rights are exercised in Czech Republic in the facilities of Work and Road Safety Corporation s.r.o. and cover the repair of the device or its replacement with the new one.
11. The warranty does not exclude, limit or suspend any rights of the Purchaser stemming from the regulations regarding a warranty for defects in sold goods.

*Before contacting the service, please check if the solution to your problem is not already provided in the user manual.*

**Serial No.:**

**Place and date of sale:**

**Seller Stamp:**

## SERVICE, CALIBRATIONS, COMPLAINTS

Do you think your breathalyzer needs maintenance? Register online and you will be able to send the breathalyzer to us completely free of charge!

**The online ticket system is nothing but benefits!**

**CAUTION - CALIBRATION OF THIS BREATHALYZER IN THE PERIOD OF 12 MONTHS FROM THE DATE OF PURCHASE IS FREE OF CHARGE (ONLY AT THE AUTHORIZED SERVICE).**

Free shipping of the breathalyzer to the service!

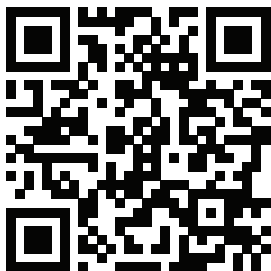
Saving your time.  
You will get everything done in a couple of minutes.

Faster service delivery - we treat online applications as a priority.

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